Spengler

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USER'S MANUAL

MANUAL

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KÄYTÖÖPÄÄS

KASUTUSJUHEND

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**MODE D'EMPLOI**

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The lyre is designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal [fig. 1]. The binaural tube spring can be adjusted for improved comfort and better adjustment to your morphology.

To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over [fig. 2]. To reduce the tension, saisir les branches de la lyre au niveau des embouts et les croiser [fig. 3].

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Thank you for choosing the PULSE/DUAL PULSE stethoscope.  
Choosing Spengler's instrument is trusting the experience and recognized excellence since 1907.

PULSE/DUAL PULSE is the result of many years of research and development aiming at providing the best quality, design and comfort to health professionals. This medical device is exclusively manufactured in accordance with the Directive 93/42 EEC, relating to medical devices and is subject to quality controls.

**INSTRUCTIONS FOR USE**

Please read carefully the user manual before using the product and keep it safe for further reference.

**INTENDED MEDICAL USE**

PULSE/DUAL PULSE is a professional equipment (can be used by trained health professionals) for medical diagnostic. It may be used for heart auscultation, lungs... as well as measuring the blood pressure.

**Caution:** This device must be used according to the instructions

**INSTRUCTIONS FOR USE**

Fitting and adjusting the binaural tubes

The binaural tubes are designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal [fig. 1].

The tension of the spring of the lyre can be adjusted for improved comfort and better adjustment to your morphology.

To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over [fig. 2].

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**Chestpiece selection (double-sided model)**

To select the desired chestpiece, gently tap this latter's membrane with the tip of your finger. If there is no sound, rotate the chestpiece through 180° until a click is heard.

**Using the large chestpiece**

The large chestpiece (diaphragm mode) is used to listen to highfrequency sounds.

**Using the small bell chestpiece (double-sided model)**

The small bell chestpiece is used to listen to low-frequency sounds.

**MAINTENANCE**

Earpiece and diaphragm replacement

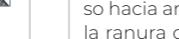
**Earpieces:**  
Check regularly that the earpieces are screwed on tight in order to ensure good acoustic quality. To replace the earpieces, remove them and screw on new ones.

**Diaphragm:**  
The diaphragm is held in place by the chill ring. To remove the diaphragm, grip the chill ring between thumb and index finger and pull to remove it from the chestpiece [fig. 4].

Replace the diaphragm on the chestpiece, printed side up, and align the edge of the chill ring with the chestpiece groove. Using your thumbs, attach the chill ring to the chestpiece circumference [fig. 5].

**UPKEEP AND CLEANING RECOMMENDATIONS**

Regular cleaning of your stethoscope and in particular of the earpieces, ensures optimum acoustic performance.

- Do not immerse the stethoscope in liquid.
- Do not sterilise.
- Clean the stethoscope with a cloth soaked in 70% isopropyl alcohol or moderately soapy water.

- Unscrew the earpieces before cleaning them thoroughly by soaking, then drying them. Ensure that no dirt remains inside.
- Keep the stethoscope away from excessive heat or cold, oils and solvents.
- Avoid exposing the stethoscope to direct sunlight.

**GARANTIE**

The PULSE/DUAL PULSE stethoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).

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**Accessories included**

- Pair of flexible TPE screw-on earpieces (small) matching tube colour

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