LITESTICK®

Lampe stylo de poche
Pocket penlight

FR

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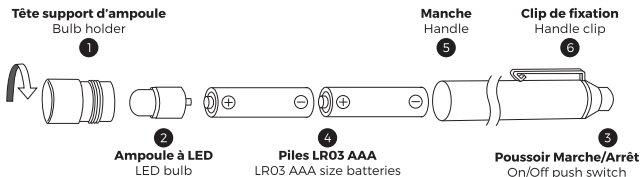
EN

This medical device is exclusively manufactured in accordance with the regulations in force, relating to medical devices and is subject to quality controls.

DESCRIPTION

Shock-resistant aluminium casing and handle clip ⁵.

Battery handle equipped with a LED bulb (3.0 V) ².

Accessories included: 2 LR03 AAA size alkaline batteries.

INSTRUCTIONS FOR USE

Inserting the batteries ⁴

Unscrew the bulb holder and insert the two batteries into the handle making sure they are aligned correctly (+) and (-).

On/Off procedure ³

Press the on/off black push button to switch the light on intermittently. Release the button to switch the light off. To switch on the light permanently, press the upper part of the handle clip ⁶.

MAINTENANCE

Replacing the batteries

Unscrew the handle from the bulb holder, remove the low batteries and replace them with two new LR03 AAA size alkaline batteries, making sure they are aligned correctly (+) and (-).

Replacing the bulb ¹

Unscrew the bulb holder from the handle and remove the bulb from the holder by pushing it from the outside towards the inside. Then insert the new bulb into the holder by pushing it forwards.

WARNING

- Do not soak, do not immerse into liquids.
- Avoid prolonged exposure to direct sunlight and to extreme temperatures.
- Place beyond children's reach.
- Remove batteries if the equipment is not to be used for a long time.

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Spirit Medical Co., Ltd

2F, No. 68, Sec. 2, Guo Dao Rd., Luzhou Dist, New Taipei City, Taiwan

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ul sw. Szczepana 20a, 61-465, Poznan, POLAND

Date de 1^{re} marquage CE : 04/11/2002

1^{re} CE marking date: 2002/11/02

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SPENGLER SAS