# magister II

<div> <div><span><span></span></span></div> <div><b>Stéthoscope</b></div> </div> <div> <div><span><span></span></span></div> <div><b>Stethoscope</b></div> </div> <div> <div><span><span></span></span></div> <div><b>Fonendoscopio</b></div> </div> <div> <div><span><span>سنتاعة</span></span></div> <div><b>سنتاعة طبيب</b></div> </div>	<div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div>	<div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div> <div> <div><span><span></span></span></div> <div><span></span></div> </div>
---	---	---

DESIGNATION / DENOMINATION / التسمية	RÉFÉRENCE / REFERENZE / الرافق / المرجعي
Stethoscope MAGISTER II Adulte simple pavillon Silver <p>MAGISTER II adult stethoscope single chestpiece - Silver</p> <p>Estetoscopio MAGISTER II para adultos simple campana plata</p> <p>MAGISTER II Erwachsene Einzelkopfstethoskop - Silber</p> <p>Stetoscopio a padiglione singolo MAGISTER II - Argentato</p> <p>MAGISTER II stethoscoop voor volwassenen met één kop - Zilver</p> <p>MAGISTER II aikuinen yhden pään stetoskooppi - Hopea</p> <p>Täiskasvanute ühe peaga stetoskoop MAGISTER II - Must</p> <p>MAGISTER II saugusugusjuhi vienoš galvos stetoskopas - Sidabrinė</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 250

Stethoscope MAGISTER II Adulte simple pavillon Black <p>MAGISTER II Adult stethoscope single chestpiece - Black</p> <p>Estetoscopio MAGISTER II para adultos simple campana negro</p> <p>MAGISTER II Erwachsene Einzelkopfstethoskop - Schwarze</p> <p>Stetoscopio a padiglione singolo MAGISTER II - Nero</p> <p>MAGISTER II stethoscoop voor volwassenen met één kop - Zwart</p> <p>MAGISTER II aikuinen yhden pään stetoskooppi - Musta</p> <p>Täiskasvanute ühe peaga stetoskoop MAGISTER II - Must</p> <p>MAGISTER II saugusugusjuhi vienoš galvos stetoskopas - Juodas</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 260
---	---------

Stethoscope MAGISTER II Adulte double pavillon Silver <p>MAGISTER II Adult stethoscope dual chestpiece - Silver</p> <p>Estetoscopio MAGISTER II para adultos doble campana plata</p> <p>MAGISTER II Erwachsene Doppelkopfge - Silber</p> <p>Stetoscopio doppio MAGISTER II per adulti - Argentato</p> <p>MAGISTER II Stethoscoop Volwassen dubbele vlak - Zilver</p> <p>MAGISTER II stetoskooppi aikuisten kaksoislippu - Hopea</p> <p>MAGISTER II stetoskooppi täiskasvanute topellipp - Hõbe</p> <p>MAGISTER II saugusugusjuhi dvigubva vēlviela - Sidabrīnē</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 251
---	---------

Stethoscope MAGISTER II Adulte double pavillon Black <p>MAGISTER II Adult stethoscope dual chestpiece - Black</p> <p>Estetoscopio MAGISTER II para adultos doble campana negro</p> <p>MAGISTER II Erwachsene Doppelkopfge - Schwarze</p> <p>Stetoscopio doppio MAGISTER II per adulti - Nero</p> <p>MAGISTER II Stethoscoop Volwassen dubbele vlak - Zwart</p> <p>MAGISTER II stetoskooppi aikuisten kaksoislippu - Musta</p> <p>MAGISTER II stetoskooppi täiskasvanute topellipp - Must</p> <p>MAGISTER II saugusugusjuhi dvigubva vēlviela - Juodas</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 261
--	---------

Stethoscope MAGISTER II double pavillon Coral <p>Dual MAGISTER II stethoscope dual chestpiece Coral red</p> <p>Estetoscopio MAGISTER II double campana Coral</p> <p>Stethoskop MAGISTER II Erwachsene Doppelkopf Koralle</p> <p>Stetoscopio MAGISTER II Adulta testina doppia corallo</p> <p>Stethoscoop MAGISTER II volwassenen dubbel borststuk Coral</p> <p>Stetoskooppi MAGISTER II aikuisille, kaksoispuolinen rintakappale Korallirintapaininen</p> <p>Stetoskoop MAGISTER II täiskasvanutele, kahepoolese kuuluariga, korallipüanane</p> <p>MAGISTER II pieaugusugusjuhi dubuuta galvina, korallij</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 252
--	---------

Stethoscope MAGISTER II double pavillon Myrtille <p>Dual MAGISTER II stethoscope dual chestpiece Blueberry</p> <p>Estetoscopio MAGISTER II double campana Arandano</p> <p>Stethoskop MAGISTER II Erwachsene Doppelkopf Blaubeere</p> <p>Stetoscopio MAGISTER II Adulta testina doppia mirtillo</p> <p>Stethoscoop MAGISTER II volwassenen dubbel borststuk Myrtille</p> <p>Stetoskooppi MAGISTER II aikuisille, kaksoispuolinen rintakappale Sininen</p> <p>Stetoskoop MAGISTER II täiskasvanutele, kahepoolese kuuluariga, tumesinine</p> <p>MAGISTER II pieaugusugusjuhi dubuuta galvina, mellelune</p> <p>السنتاعة البتابة الكتابة رافق العلام العلام الالوانى</p> <p>MAGISTER II</p>	507 262
--	---------

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USER'S MANUAL	EN
Thank you for choosing the MAGISTER II stethoscope. Choosing Spengler's instrument is trusting the experience and recognized excellence since 1907. MAGISTER II is the resulting from many years of research and development aiming at providing quality, design and comfort to health professionals. This medical device is exclusively manufactured in accordance with the regulations in force, relating to medical devices and is subject to quality controls.	

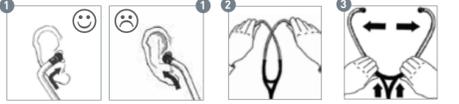
**INSTRUCTIONS FOR USE**  
Please read carefully the user manual before using the product and keep it safe for further reference.

**INTENDED MEDICAL USE**  
The MAGISTER II is a professional equipment for aid in the diagnosis. It is an acoustic instrument used for auscultation to listen to sounds within the human body in order to make a medical diagnosis for example during heart, lungs and other organic noises auscultation as well as measuring the blood pressure.

**INDICATIONS**  
MAGISTER II stethoscope is a medical device Class I, for aid in the diagnosis, intended for health care professionals, trained for its use. It is used to listen to sounds within the body for gathering information about patient condition and to measure blood pressure for all types of patients: adults, children and newborns.

**CONTRAINDICATIONS**  
- Do not use on injured skin.  
- Do not used in case of instruments material allergies.

**INSTRUCTIONS FOR USE**  
**Fitting and adjusting the binaural tubes**

The binaural tubes are designed to automatically adjust to your ears' listening angle. Ensure that the branches of the binaural tubes are pointing forwards, thus matching the direction of the auditory canal [fig. 1].  
The binaural tube spring can be adjusted for improved comfort and better adjustment to your morphology.

To increase the spring tension, grasp the branches of the binaural tubes at the level of the earpieces and cross them over [fig. 2].

To reduce the spring tension, grasp the branches at the base of the binaurals and gradually move them apart to complete extension [fig. 3].

**Selecting chestpiece for MAGISTER II DUAL**  
Thanks to the innovative chest-piece indicator (red dot), you can see which side of the dual head is activated.

**Using the multi-frequency floating diaphragm**  
The MAGISTER II stethoscope enables you to perform various types of auscultation. The multi-frequency floating diaphragm can be used to listen to both low and high frequency sounds without the need to flip the chestpiece.  
• **Low frequencies**  
: Apply very mild pressure to the chestpiece. The diaphragm is held in place by a ring, enabling low-frequency sounds to resonate.  
• **High frequencies**  
: Apply firm pressure on the chestpiece. The diaphragm makes contact with a circular edge, thus allowing high-frequency sounds to be heard.  
To alternate between listening to high and low-frequency sounds, simply alter the pressure applied to the chestpiece.

**Using the small bell chestpiece**  
The small bell chestpiece allows listening to low frequencies sounds.

Chest-piece	Single head	Dual Head
Material	Stainless steel	Stainless steel
Weight	81g	105g
Diametro del lado grande	45,36mm	45,36mm
Ø Diaphragm	45,36mm	45,36mm
Ø Bell	-	34,38mm
Floating membrane		
Material	Epoxy resin Fiber glass	Epoxy resin Fiber glass
Ø Diaphragm	44mm	44mm
Ø Bell	-	34,8mm
Non-chill ring		
Material	Thermo-propylen rubber	Thermo-propylen rubber
Ø Diaphragm	47,3mm	47,3mm
Ø Bell	-	37,5mm
Binaurals and other parts		
Tubing length	61 cm	61 cm
Binaurals material	Aluminium	Aluminium
Spring material	Stainless steel	Stainless steel
Tubing material	PVC	PVC
Ear tips material	Silicon	Silicon
<b>Total length</b>	<b>78 cm</b>	<b>78 cm</b>
<b>Accessories included</b>	<b>2 pairs of ear-tips</b>	

**WARNINGS**  
- Do not immerse the stethoscope in water/liquid.  
- Do not use if the packaging is damaged.  
- Do not sterilize.

**SIDE EFFECTS**  
None

**MATERIOVIGILANCE**  
All serious incidents related to the use of a medical device should be notified to the manufacturer and the responsible authority of the Member State where the user and/or the patient live.

**STORAGE AND HANDLING CONDITIONS**  
- Keep the stethoscope away from excessive heat or cold.  
- Avoid exposing stethoscope to direct sunlight.

**MAINTENANCE**  
Earpiece and diaphragm replacement  
• **Earpieces**  
: Regularly ensure that the ends are properly clipped/screwed on to preserve good acoustic quality. To replace them, pull on the ends then replace them by pushing the new ends into place.

• **Diaphragm**  
: The diaphragm is held in place by the chill ring. To remove the diaphragm, grip the chill ring between thumb and index finger and pull to remove it from the chestpiece [fig. 4].  
Replace the diaphragm on the chestpiece, printed side up, and align the edge of the chill ring with the chestpiece groove. Using your thumbs, attach the chill ring to the chestpiece circumference [fig. 5].

If the device seems damaged or if the sounds are not audible anymore, do not use the device. Contact the after sales department.

**UPKEEP AND CLEANING RECOMMENDATIONS**  
Regular cleaning of your stethoscope and in particular of the earpieces, ensures optimum acoustic performance.  
- Clean the stethoscope with a cloth soaked in 70% isopropyl alcohol or moderately soapy water.  
- Unswrew the earpieces before cleaning them thoroughly by soaking, then drying them. Ensure that no dirt remains inside.

**DISPOSAL**  
Dispose of the device at the specific collection point according to local regulation.

**GUARANTEE**  
The MAGISTER II stethoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).  
- Chestpiece, binaural tube assembly: 5 years  
- Earpieces, diaphragm and chill rings: 1 year

The guarantee does not apply in the event of use non-compliant with instructions, improper use, modifications made by yourself or by a unauthorised third party, accidental damage and normal wear.  
During the guarantee period, any stethoscope parts that present manufacturing defects shall be replaced and/or repaired free of charge, after returning the device to Spengler's after sales service. In the event of a guarantee claim or of repairs, please return the complete stethoscope in its original packaging, accompanied by the dated invoice.

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