Spengler

SMARTLIGHT

KÄYTTÖOHJE
MODALITÀ D'USO
LIETOSANAS INSTRUKCIJA
GEBRUIKSAANWIJZING
طريقة الاستخدام

MODE D'EMPLOI
INSTRUCTION MANUAL
BEDIENUNGSANLEITUNG
MODO DE EMPLEO
KASUTUSJUHEND

OTOSCOPE SMARTLIGHT

Symboles/Symbols/Símbolos:

	Fabricant - Manufacturer- Hersteller - Fabricante - Tootja - Valmistaja - Produttore - Ražotājs - Fabrikant
	EC REP

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EN SMARTLIGHT OTOSCOPE EN

INSTRUCTION MANUAL

An instruction of otoscope for ENT specialists. It is an internally powered device. Please read through this brochure before use, and keep it handy for future reference.

SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

WARNINGS & CAUTIONS:

- Do not use the device where it may be exposed to flammable gas.
- Protect this device from water, moisture, impact, dropping, direct sunlight, heat and fire.
- Do not touch the lens directly by hand or something hard.
- Please use the bulb type specified in this instruction brochure.
- Keep the device out of the reach of unsupervised infants and Children.
- Do not disassemble or attempt to repair the device or components.

GENERAL

Description: The otoscope is a classic medical device in ENT. As a traditional diagnostic otoscope, SMARTLIGHT adopts Direct Illumination Technology. It is used to view the tympanic membrane and external ear canal to diagnose outer and middle ear pathologies.

Technical Specifications:

- Enhanced XHL Xenon/Halogen Technology
- 3X magnification for the viewing window.
- #108 bulb used.

HOW TO USE THIS DEVICE

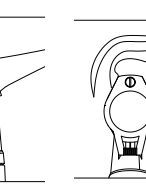
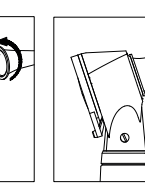
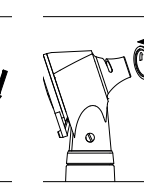
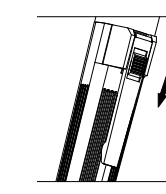
fig.1 fig.2 fig.3

- Turn on (or off) the device by pushing downwards (or upwards) as **fig.1**;
- Select a reasonable tip for patient's ear canal and push it onto the hand and screw the tip lockwise, refer to **fig.2**. Generally, the tip size of 4mm is for adult and 2.4mm is for children;
- Put the otoscope with a tip into the ear canal of patient (**fig.3**), and examine through the viewing window which is with 3X magnification.

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MAINTENANCE

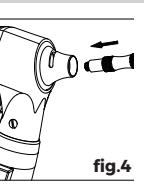
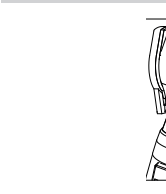
fig.4 fig.5

Changing Spare Parts:
1. Changing the bulb (**fig.4**)
Turn off the power, then use screwdriver, screw the old bulb in the metal support and replace with new one.

ATTENTION :

- Please use the specialized bulb (Model : #108) for this device.
- After the bulb burns out, the temperature will remain high for approximately 5 minutes. Please wait for a moment to avoid injury.

2. Changing the batteries (**fig. 5**)

• Screw the head off the handle.- Remove the old AA batteries and replace with new ones. Ensure the anode of the battery points to the head of the device.
- Screw the head back on the handle.

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Warnhinweise:

- AA alkaline batteries are recommended.
- Replace all batteries simultaneously and do not use alkali manganese batteries with lithium as bulb life will be reduced.
- Do not leave worn batteries in the handle.
- Please take out batteries from the device if not in use for a prolonged period.
- Batteries should be disposed of in accordance with local regulations.

Cleaning and Disinfection:

- Clean the surface of device with 75% alcohol solution at least once a week, avoiding the magnifying glass (viewing window) in the head. And corrosive or polishing detergent is Prohibited.
- Disinfect the tips of otoscope with saturated steam for 20 minutes at 134°C.

ATTENTION :

Do not soak, autoclave or let water dip into the device!

PERIODIC SAFETY CHECKS

Preventive inspection and maintenance to be performed including the frequency of such maintenance. The following safety check should be performed at least every 12 months by a qualified person who has adequate training, knowledge, and practical experience to perform these tests.

- Inspect the equipment and accessories for mechanical and functional damage.
- Inspect the safety relevant labels for legibility.
- Verify that the device functions properly as described in the instructions for use.

SPECIFICATIONS

Illumination	Direct Illumination
Luminance	24 Lumens
Color temperature	3050K ±5%
Magnification	3 x
Tip Sizes	2,4 mm, 3 mm, 4 mm, 5 mm
Bulb	Specified Model: #108 2,5 v, 0,75 A, 3050 K, 25 hrs
Batteries	2 AA alkaline batteries
Operation	Température 10 °C - 35 °C Humidité 30 % - 75 % Air pressure 700 hPa -1060 hPa
Stockage & Transport	Température -20 °C - 70 °C Humidité 10 % - 90 % (without condensation) Air pressure 500 hPa -1060 hPa
Guarantee	2-years (from date of purchase)
Standards	RDM (UE) 2017/745, Class I IEC 60601:2012, IEC 60601-1-2:2014

ELECTRICAL CLASSIFICATION

Equipment Classification with respect to protection from electric shock: Internally powered equipment. Degree of protection from electric shock: TYPE B. Equipment not suitable for use in the presence of flammable mixtures. Yes. Mode of operation: intermittent operation.

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