Spengler

TEMPO DUO

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Guangzhou Berrcom Medical Device Co., Ltd
Address: No.38 Huanzhen Xi Road, Dagang Town,
Nansha, 511470 Guangzhou, PEOPLE'S REPUBLIC OF CHINA

Welkang Ltd
Address: The Black Church, St. Mary's Place,
Dublin 7, D07 P4AX, Ireland
Tel: +353(1)4433560
Email: AuthRep@CE-marking.eu
Web: www.CEmark.com

TEMPO DUO

15

EN

Ear and forehead Thermometer User's manual

CE
0123

IMPORTANT

Read this manual completely before using the Spengler Ear and forehead Thermometer.

Save these instructions.

EN

1. Caution	p. 17
Warning	p. 17
Attention	p. 18
Suggestion	p. 18
2. Indications	p. 18
3. Description	p. 19
4. Features	p. 20
5. Operating Principle	p. 21
6. Operating Instructions	p. 22
Insert Batteries	p. 22
Operating Procedure	p. 22
7. Technical characteristics	p. 25
8. Contents	p. 26
9. Maintenance	p. 26
10. Trouble shooting	p. 27
11. Warranty Card	p.56

- **VERSION FRANÇAISE** P. 3
- **ENGLISH VERSION** P. 15
- **VERSIÓN ESPAÑOLA** P. 28

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Warning

1. It is dangerous to make self-determination and take therapy accordingly merely on the basis of the testing results. So it is suggested to do following doctor's guidance.
2. Do not touch with hand or blow with mouth on the infrared sensor. Any break or dirt on the infrared sensor would probably result in incorrect measuring.
3. Clean the infrared sensor with a cotton bud lightly moistened with 95° alcohol.
4. Keep out of the reach of children.
5. Please wait a minute till the ear reaches a stable temperature, since the ear temperature is rather low the moment you get home.
6. Do not operate immediately from a significant change in the ambient temperature to allow the thermometer acclimatise to the ambient temperature for 30 minutes before using it.
7. Avoid dropping, crashing and severe shock to the device.

1. CAUTION

EN

Attention

1. Do not insert the probe into the ear canal by force. And stop using when there is any abnormal sensation during performance, like hurt etc.
2. Not applicable for the people bearing illness like otitis externa and tympanitis etc.
3. Do not use when the ear is wet after swimming and bathing.
4. Do not throw battery into fire.

Suggestion

- Do demonstrate your temperature is taken by using ear and forehead thermometer when you tell the doctor your body temperature.
- Do not hit, drop, step and shake the main unit.
- Do not dismantle, repair and retrofit the main unit.
- Do take care of liquid inward into inside (such as alcohol, drops, hot water etc.) since the device is not waterproof design.
- Do keep in clean and dry place.
- Do not repair it by yourself but reach our distributor on discovering any problem.
- Do not operate in electromagnetic interference environment.
- Do follow the local legislation and regulations to dispose the device and batteries.



2. INDICATIONS

The Spengler TEMPO DUO Thermometer is indicated for taking a body temperature from the ear and forehead.

3. DESCRIPTION

19

EN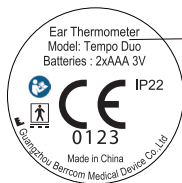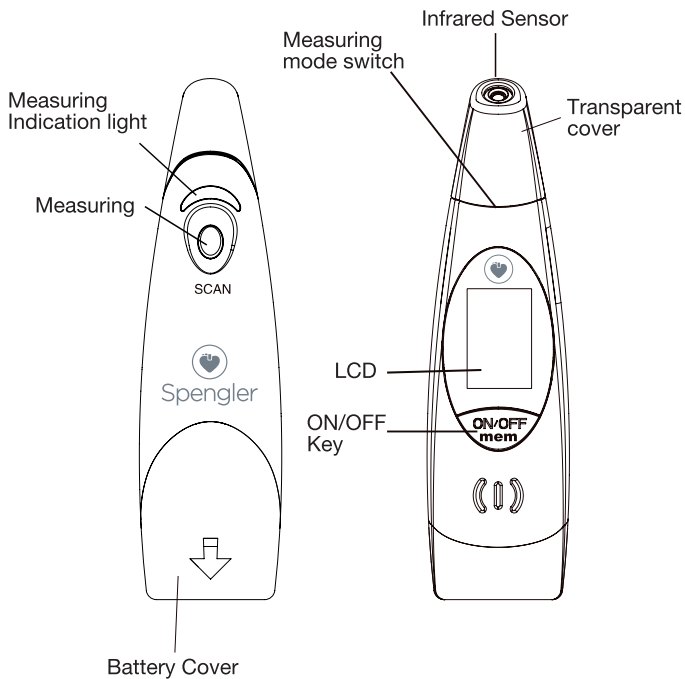


Rating label

EN

1. Measuring Scope: 32.0°C ~ 42.9°C
2. LCD display: three color backlit display.
In Ear Thermometer mode
Green ≤ 37.2 °C
Orange 37.3 C° $\leq T \leq 37.9$ C°
Red ≥ 38.0 C°
In Forehead Thermometer mode
Green: Normal temperature
Orange: Temperature to watch
Red : Fever
3. Data memory: memory of the last 10 measurements for analysis and comparison.
4. Automatic power-off: The Spengler TEMPO DUO thermometer turns off automatically after 75 seconds of inactivity.
5. Outlook design: Compliance with the ergonomics.

Ear temperature: Ear tympanic blood vessel connects with the hypothalamus, which is the temperature control center in human cerebral; therefore, by means of testing the ear tympanic temperature, it can figure out the accurate body temperature of human being. Spengler TEMPO DUO is such a product to detect body temperature of human being by means of temperature taking on ear tympanic through infrared sensor.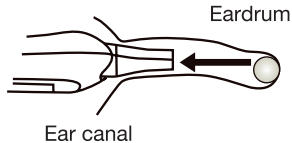

Forehead Temperature: The infrared sensor receives the heat energy generated by artery blood, the device figures out the precise body temperature of human being.

Reference normal temperature of human being by World Health Organization (WHO):

Ear temperature	35.8°C ~ 38°C
Axillary temperature	34.7°C ~ 37.3°C
Oral temperature	35.5°C ~ 37.5°C
Rectal temperature	36.6°C ~ 38°C


Human body temperature differs from one person to another, even for the same person at different body site and different time. Generally speaking, the majority of people's ear temperatures are much higher than the temperature in oral and axillary. But also there is the opposite condition. To get correct judgment over fever condition, it is necessary to have a good understanding of the normal body temperature of your family members.

6. OPERATING INSTRUCTIONS

EN

Insert Batteries

Install a couple of AAA batteries and pay attention to the polarity (see below chart).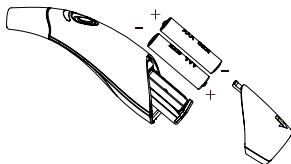


Operating procedure

1. Power on

Press “ON/OFF” key to start the device and take temperature when the screen shows up with “0.0°C” or “0.0°F”.

Putting the transparent cover on the top is for testing forehead temperature and taking it down is for testing ear temperature.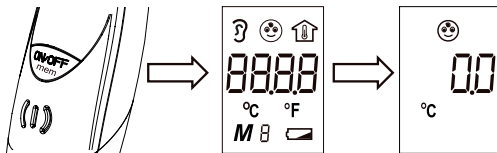

2. Ear temperature measurement

Take the transparent cover off, and then the screen will show up with the sign of ear “ E ”, which is for the ear temperature mode. Pull the ear edge, fix the head, insert the probe into ear canal, point to the eardrum, then press the “measuring” key will give a detecting signal like “-----” and the measuring indication light will blink. In five seconds, make a beep sound when the measuring will be closed.

6. OPERATING INSTRUCTIONS

23




EN

Make sure the probe aims at the eardrum when inserting it into the ear canal; otherwise, the result will be lower than the actual body temperature.

✔ 36.8°C

⊗ Measures lower

⊗ Measures lower

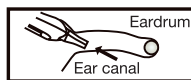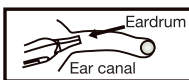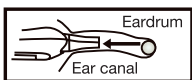Approach for straight the ear canal:

Newborn under 12 months:
pull in outward rear direction.

Child above 12 months and the adult:
pull in outward upper direction.

3. Forehead Temperature measurement

Put the transparent cover on the top then the screen will show up with a sign of forehead like ☺, which means forehead temperature mode press the “Measuring” key will give a detecting signal like “-----” and the measuring indication light will blink. In five seconds, make a beep sound when the measuring will be closed.

Make sure the probe is appressed to the middle forehead during testing (See below chart).

Forehead Temperature

Forehead temperature by ear and forehead thermometer is the appraisal measuring value.

6. OPERATING INSTRUCTIONS



EN

4. LCD Indication

 Temperature reading by ear thermometer in ear temperature mode.

 Spengler TEMPO DUO LCD Displays the different color to indicate the body temperature in forehead temperature mode.

5. Choosing the unit: Pressing the “ON/OFF” key and the “measuring” key unit for five seconds above under the testing mode will enter unit shifting mode between C°/F°.

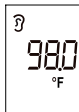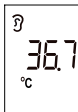6. Memory: Press the “ON/OFF mem” key under the testing mode till the screen shows up with Mx, the “x” of which means the group number of memory, and then displays the measuring value. “M0” is the latest testing value, and “M9” is the earliest one.

7. Ambient temperature measurement: If there is no key input within 10 seconds after body temperature testing, the device will show up with the current ambient temperature.

6. OPERATING INSTRUCTIONS

25

NE

- Power off: press the “ON/OFF” key in five seconds to switch off device (The Spengler TEMPO DUO turn off automatically after 75 seconds of inactivity.)
- Allow a ten seconds interval between two measurements. When the LCD display the room temperature after last measuring.

7. TECHNICAL CHARACTERISTICS

Product name:	TEMPO DUO
Type:	Ear and forehead thermometer
Voltage:	DC3V (a couple of AAA batteries)
Power consumption:	0.1 W
Battery life:	20 000 times of measurement
Measuring principle:	Infrared measurement
Measuring range:	32.0 °C ~ 42.9°C
Display resolution:	0.1 °C
Precision:	± 0.2 °C
Display mode:	LCD screen
Data memory:	memorize 10 measures
Automatic power-off:	75 seconds
Unit size:	126 mm (L) × 35 mm (W) × 45 mm (H)
Unit weight:	66g (including batteries)
Normal using conditions	
Operation temperature:	16 °C ~ 35 °C (if the ambient temperature is out of these range (16°C-35°C) the LCD will displays “Err”.)
	Relative humidity ≤ 85 %
Storage conditions:	Ambient temperature -25°C ~ 55°C
	Relative humidity ≤ 85 %

EN

- 1 main unit
- 1 user manual
- 2 batteries (AAA)

9. MAINTENANCE

Please do keep maintenance and care with this product after use.

1. Do remove the dirt on the device with a soft and dry cloth.
 - > Do not wash in water or the detergent, diluent and volatile oil embracing abrasive elements.
2. Please mildly clean the infrared sensor with cotton bud.
 - > Do not clean with a tissue or paper napkin.
3. Store thermometer in a dry location free from dust and contamination and away from direct sunlight.
4. Do take out of the battery when there is long period of time of non-using.
5. Do replace batteries when there is a low voltage icon “” sign. Do not mix the old ones with new ones.
6. The device was developed and built with high performance sensor and high quality components to insure long term reliable operation. Should any problem arises, you could call your local retailer.

Problems	Causes	Solutions
No display hen turning on the power	Battery drained	Replace 2 (AAA) batteries
	Put batteries into the wrong polarity location	Keep the polarity correct with indicated on battery box
Measurements is lower	The wrong method of measuring	Measuring according to the correct user manual (see page 23)
	Dirt in the ear canal or on the sensor	Measuring after clean away the dirt
Variation on continuous measuring temperature	Small measure interval	Each measuring interval should be more than 10 seconds

Distributed by SPENGLER SAS
30 rue Jean de Guiramand - 13290 Aix en Provence - FRANCE

Guangzhou Berrcom Medical Device Co., Ltd
Address: No.38 Huanzhen Xi Road, Dagang Town,
Nansha, 511470 Guangzhou, PEOPLE'S REPUBLIC OF CHINA

Welkang Ltd
Address: The Black Church, St. Mary's Place,
Dublin 7, D07 P4AX, Ireland
Tel: +353(1)4433560
Email: AuthRep@CE-marking.eu
Web: www.CEmark.com

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